

R82°



TM mustang

Motivating the child to walk

The user is placed at the centre of the Mustang which provides stability of the upper body and enables the user to focus energy on walking



Develop a more natural walking pattern

The Mustang has easy and steady angle adjustable centre bar. A slight forward leaning position, combined with built-in suspension, stimulates the natural walking pattern



Easy to access

The Mustang has been constructed with accessibility in mind. The U-shaped frame and wrap around support, allows for easy access and transfer



Easy to store and carry

The foldable frame makes storage and transport of the Mustang easy



The Mustang is a walking aid for disabled children, who need supplementary assistance with their daily walkina.

The Mustang provides excellent and snug support for the upper body while also keeping the child's pelvis gently supported. The upright, slightly forward leaning position stimulates the walking function and encourages the child to participate in daily activities.

The Mustang can be used by e.g. CP children and others with lack of balance and impairment of their motor function. The Mustang is developed as a basic anterior walker but can also be used as a posterior gait trainer, depending on the child's level of ability.

The Mustang is available in four sizes and suitable for children between the ages of 2 - 18 years.









14-60 cm 37-125 cm max. 130 cm max. 80 kg





STANDARD CONFIGURATION

Standard

As standard the Mustang is delivered with lockable wheels. The Mustang is available in 4 sizes



Size 2, 3, 4 with larger wheels

Frame colour Available in 4 colours









Size 1 = Yellow

Size 2 = Blue

Size 3 = Red

Size 4 = Blue

Wheels

The Mustang is as standard delivered with various adjustment possibilities on all four wheels



Directional stabilizer



Tension adjustable wheels



Wheel brake on all 4 wheels

Wheel stops

The unique on/off anti-reverse wheel stops are very easy to lock and unlock




Lock



Release



MEASUREMENTS

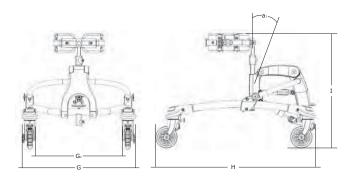

| Measurements | | |
|--|-----------------------|--------------------------------------|
| | Size 1 | Size 2 |
| Width, total (G) | 52 cm (201/4") | 63 cm (24 ") |
| Width, inside frame (G1) | 38 cm (14 ") | 45 cm (17 ") |
| Length, total (H) | 75 cm (29¼") | 90 cm (35") |
| Height, total (I) | 65 cm (25½") | 80 cm (31¼") |
| Seat height above floor (seat = accessory) | 17-35 cm (6 -13 ") | 33-50 cm (12 -19 ") |
| Width between handles (handles = accessory) | 14-35 cm (5 -13 ") | 14-35 cm (5 -13 ") |
| Height from floor to handles (handles = accessory) | 37-58 cm (14 -22 ") | 51-71 cm (20-27 ") |
| Angle range of centre bar (a1) | 30° | 30° |
| Turning radius | 0° | 0° |
| User height, floor to axil | 38-65 cm (14 -251/4") | 60-80 cm (23 -31¼") |
| Weight, standard without accessories | 10 kg (22 lb) | 14 kg (31 lb) |
| Max. load / user weight | 30 kg (66 lb) | 40 kg (88 lb) |
| | | |
| | Size 3 | Size 4 |
| Width, total (G) | 65 cm (25 ") | 70 cm (27 ") |
| Width, inside frame (G1) | 50 cm (19 ") | 56 cm (22") |
| Length, total (H) | 100 cm (39") | 110 cm (43") |
| Height, total (I) | 105 cm (41") | 130 cm (50 ") |
| Seat height above floor (seat = accessory) | 45-68 cm (17 -26 ") | 62-84 cm (24-32 ") |
| Width between handles (handles = accessory) | 14-60 cm (5 -23 ") | 14-60 cm (5 -23 ") |
| Height from floor to handles (handles = accessory) | 65-90 cm (25 -35") | 81-125 cm (31 -48 ") |
| Angle range of centre bar (a1) | 0-25° | 0-25° |
| Turning radius | 55 cm (21 ") | 62 cm (24 ") |
| Turriing radius | 33 GIT (ET) | |
| User height, floor to axil | 70-95 cm (27 -37") | 87-130 cm (34-50 ") |
| | | 87-130 cm (34-50 ") 19 kg (42 lb) |

The Mustang is measured as standard delivered and with the center bar in vertical position. Different settings may result in another set of measurements. To accomplish some of the measurements above, a slight adjustment might be required. We supply various types of accessories, which is why the weight is listed without.



ACCESSORIES

Chest support

8698x0

For support of the upper body

size measurements

O: 47-64 cm

2 O: 67-78 cm

O: 87-114 cm

O: inner circumference



To eliminate the gap when width adjusting the chest support

measurements size H: 11 x W: 5½ cm one



Cover 8698x0-x60 For added comfort

size measurements

1 H:81/2 x W:25 cm

2 H:12 x W:38 cm

H:15 x W:49 cm



Hand bar

86936x Provide hand control For anterior use

measurements size S H:11 x W:21 cm H:11 x W:32 cm



Abdominal support 869260

Provides additional anterior pelvic control

measurements one H:13 x W:19 cm



Arm rests, anterior

86935x

Provide forearm support. Helps the child to stabilise the trunk and head

Two sizes



Sacrum support

Helps to stabilizes the pelvis. Requires seat 8691X0

measurements size

H:18 cm (from seat) D:13-21 cm (from seat)

H:20½ cm (from seat) D:161/2-241/2 cm (f.seat)



Arm prompts, anterior or posterior

8693xx

Helps the child to stabilize the trunk and head

Three sizes



Hip support, small 869220 or 869221

Helps to stabilise the pelvis. Requires seat 8691X0

size measurements one W:181/2-251/2 cm



Hip support, large 869230 or 869231

Helps to stabilise the pelvis. Requires seat 8691X0

measurements size one W:251/2-321/2 cm







ACCESSORIES

Head support system 861424-3 and 861424-4

size measurements 1 Rod = L:20 cm2 Rod = L:33 cm



Seat

8691x0. Provides a resting position that a child can use when fatigued. Used for mounting sacrum or hip supports

size measurements H:17½ x D:25½ cm 2 H:17½ x D:30½ cm



Connecting rod

869226 Mounted between hip and head support

One size



Sling seat

86957x Provides partial weight bearing support

measurements size W:18 x L:24½ cm 1 W:18 x L:33 cm



Leg pad 869470 Provides knee protection

One size



Guide bar

869510 Allows caregiver to direct and assist the child's movements

measurements L: 70 cm



Storage pouch 869560

For transporting toys etc. Only for size 1+2

measurements size H:14½ x W:13 cm one



Storage bag 869565

Only for size 3+4

measurements size H: 30 x W: 39½ cm one



Leg separator 8694x0

Protects children with the tendency of scissoring their legs

Four sizes



Ankle prompts 8694xx

Guidance for correct walking pattern

measurements size L: 52½ cm 1

2

L: 60½ & 26½ cm

3 L: 70½ cm

L: 80½ cm

